

# Kodak Medical X-Ray Processor

## Step up to a KODAK Film Processor

*The KODAK Medical X-Ray Processor offers reliable automated processing at a very affordable price. This tabletop unit fits almost anywhere in your clinic, private practice, or other health facility. A simple, streamlined design provides high reliability, ease of use, and easy service maintenance.*

*This processor is a member of the KODAK Medical X-Ray product line—film, screen, cassette, processing-chemical, and processor offerings—delivering excellent performance at attractive prices. It's all part of our commitment to help every customer meet their image-quality, productivity, and service needs.*

## Image quality

Designed for general radiography applications, this tabletop unit can deliver surprising image quality for a shallow-tank processor—especially when used with Kodak Medical X-Ray Films and Processing Chemicals.

## Consistent results

Reduce artifacts and obtain consistent results with the aid of anti-crystallization and anti-oxidation cycles that help maintain clean racks and proper chemical activity when the processor is not in use.

## Films up to 35cm wide

Process sheet films up to 35 x 43 cm (14 x 17 in.)—or as small as 10 x 10 cm (4 x 4 in.). Process roll film up to 35 cm (14 in.) wide.

## Images in less than 2 minutes

View images in as little as 1 minute, 45 seconds. Compare this to 5 minutes for standard manual processing.

## Chemical, water, and energy savings

Reduce chemical use and cost with “intelligent” automatic replenishment that uses a microsensor to replenish chemicals on the basis of actual film surface processed.



Save water and energy costs with an integrated standby mode that reduces consumption when the processor is not in use.

## Easy to use and maintain

A convenient control panel enables easy filling of tanks during the warm-up cycle. Temperature setting is simple and accurate with microprocessor control. The processor automatically signals when it is ready to process the next film. Few moving parts and easy-to-service film-transport racks simplify maintenance and service.

# Kodak Medical X-Ray Processor

## Service and support

Authorized distributors of KODAK products who sell the KODAK Medical X-Ray Processor provide service and support in association with Kodak. Please contact your local distributor or Carestream Health representative for more information.

## Installation efficiency

Ease of installation means you'll be processing films quickly. Facility staff can install the processor in approximately four hours. A trained distributor requires approximately

two hours.

## Choice of optional accessories

Enhance the convenience and value of your KODAK Medical X-Ray Processor with a choice of optional accessories:

- Medical X-Ray Processor Roll Film Adapter/ 220 V, 50 Hz
- Medical X-Ray Processor Stand
- Medical X-Ray Processor Cabinet
- Medical X-Ray Processor Exhaust Kit/ 220 V, 50 Hz
- Replenishment Tanks, 25 L
- Replenishment Tanks, 12 L (included with processor)
- Replacement Motor/Driveshaft Gears (size 12, 14, 16, 17, 18)

## Specifications

### Processing cycle

105 seconds—standard cycle  
90-second cycle\*/135-second cycle capability

### Throughput

62 films/hour (35 x 43 cm/14 x 17 in.)  
@ 105-second cycle  
72 films/hour (35 x 43 cm/14 x 17 in.)  
@ 90-second cycle  
48 films/hour (35 x 43 cm/14 x 17 in.)  
@ 135-second cycle

### Dimensions

Length: 77 cm/30 in.  
Width: 59 cm/23 in.  
Height: 42 cm/16.5 in.

### Weight

Without solutions: 35 kg/77 lbs.  
With solutions: 50 kg/110 lbs.

### Electrical

Model 101: 110/120V, 50/60Hz, 15 Amp  
Model 102: 220/240V, 50Hz, 8.8 Amp  
Consumption:  
Model 101: 1.6 kW/h during processing  
(0.12 kW/h in standby mode)  
Model 102: 2.1 kW/h during processing  
(0.12 kW/h in standby mode)

### Water

Ambient: 5-30°C (41-86°F)  
Flow rate: 1.9 L/min (0.5 gal./min)  
during processing (0.0 in standby mode)

**Kodak**

Licensed Product

Carestream Health, Inc.  
150 Verona Street  
Rochester, N.Y. 14608  
U.S.A.

Carestream Health Canada Company  
6 Monogram Place  
Suite 200  
Toronto, Ontario M9R 0A1  
CANADA

The Kodak trademark and trade dress are used under license from Kodak.  
Printed on recycled paper containing 10% post consumer waste fiber, using soybean-based inks.  
M6-116 Printed in U.S.A. © Carestream Health, Inc. 2009 CAT No. 145 2762

## For More Information

To learn more or to order, contact your Carestream Health representative or call 877-865-6325, Ext 538.

→ [www.carestreamhealth.com](http://www.carestreamhealth.com)

**Carestream**   
HEALTH